



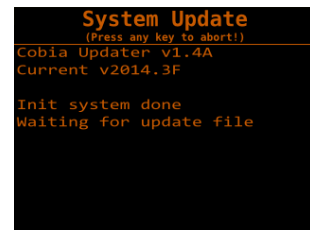


How to Install a Cobia System Update

The Cobia system update can easily be done on any computer with support for USB memories. Below is a step-by-step guide for when using a computer with Windows OS, but the procedure can be done on a computer with any OS, just as long as it has a USB port.


1. On a Windows PC, start a File Explorer window (WIN+E) and unpack the Cobia Update zip file to a local drive.
2. Start the Cobia and connect the cable between the PC and the Cobia.
3. On the Cobia, press the menu button and enter the “System Info” menu. Then, press and hold down the right arrow button for approximately 4 seconds. See images below.

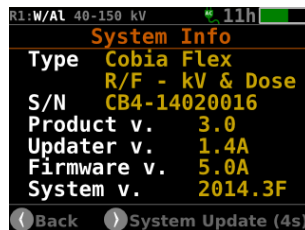

4. After a short while, a USB memory device named COBIA (as shown below) will appear in the File Explorer on the PC, and the Cobia display will show “Waiting for update file”. Now, drag-and-drop the unpacked updater file (.cobia) from the PC local drive using the File Explorer to the USB device named COBIA.

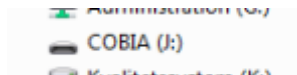

5. A counter will now start on the Cobia display during the resource file update procedure. It will take a few minutes. When the update is completed the following screen will appear:

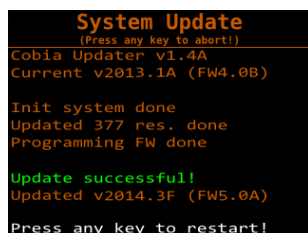

6. Press any key to launch the updated Cobia firmware.

After the update, you may want to change the language of the Cobia user interface. You will find this setting in the menu, under User Preferences.

/RTI Group