



# Product Note

## Ocean Next – News in Release v2.0.2.0

### Release information

Release date	2022-09-23
Release name	v2.0.2.0
EXE file version	1.0.2.30306
Piranha DLL (RTICmlib.dll)	3.0.1.9145
Scatter Probe DLL (rtiDeviceConnector.dll)	1.1



This is a Maintenance Release of Ocean Next.

Several bugs have been fixed and stability increased. Added the possibility to import Site-information via Excel documents.

### Connectivity

Ocean Next need to be able to communicate over the following ports: 5512 (Localhost), 5513 (Localhost), 8443 (External) to properly function.

### Database format

This version of Ocean has a new database structure compared to previous versions. When installing and running this version of Ocean it will automatically update the database.

Note that the location of the Ocean database has moved to:

"C:\Users\[WindowsUser]\AppData\Local\RTI Group\Ocean Next\ProgramData\OceanData.db"



# Product Note

## Cobia and Piranha firmware

### Piranha

This release comes with a new Firmware for Piranha. The lowest firmware that shall be used with this release of Ocean is 4.3A.

### Cobia

No new Cobia firmware version, in this release. The lowest firmware that that is recommended to be used with this version of Ocean Next is 2019.12A.

### RTI Scatter Probe

The latest Firmware version for RTI Scatter Probe is v1.2. There is no need to update to this version as this version is only applicable to the newest versions of the Scatter Probe hardware which already ships with this version.

## Known Limitations

The table below shows issues that have been verified, and valid in this release of Ocean Next.

Description
No known limitations.

\*\*\* END \*\*\*